

Tools required:
Phillips Drive Screw Driver
Tape Measure

Parts Included:
(1) Drawer Assembly
(4) 1/4-20 x 1/2" Long Phillips Head Machine Screws w/Lock Washers
(2) Bins
(4) Bin Dividers

- 1** To fasten Drawer Assembly to the Worksurface Assembly, place drawer assembly underneath the worksurface assembly.

- 2** Align the (4) hole on the arms of the drawer assembly with the (4) hole in the worksurface assembly.

Once holes are aligned, insert and tighten (4) 1/4-20 x 1/2" Long Pan Head Machine Screws.

