

Tools required:  
Phillips Drive Screw Driver  
Utility Knife/Exacto Knife  
Straight Edge  
Tape Measure

Parts Included:  
(1) Basket  
(4) #10-24 x 7/8" Long Phillips Drive Pan Head Machine Screws  
(1) .5" x 3.5"W Adhesive-Backed Foam Strip

Each Clio cart is equipped with a cable management enclosure. On the enclosure there is an adhesive cover which can be peeled off to expose fastening holes for column-mounted accessories.

Please follow the next steps to cut the cover:

**1 For top-mounted accessories:**

From the top of the cover, measure down 2.5". Mark with pencil and align marks with straight edge. With Utility Knife or Exacto Knife cut cover to expose 2 screws and 2 fastening holes.

Remove top (2) screws.

**2 Place foam tape on the back of hanging baskets, centered and just below bottom tear-drop holes.**

**3 Insert and tighten (4) Silver #10-24 x 3/8" Long Phillips Drive Pan Head Machine Screws through the holes of the Utility Basket and into the holes of the Cable Management Enclosure.**

