

Tools required:
Phillips Drive Screwdriver

Parts Included:
(1) Adjustable VESA Bar Code Holder

1 To Install Adjustable VESA Bar Code Holder (BCH-14):

Remove Monitor from LCD Monitor Support (FSA-5).

Insert the top (2) Long Phillips Drive Flat Head Machine Screws (16mm) into monitor.

Rest VESA Plate on the back of the Monitor. Slide the Adjustable VESA Bar Code holder up and into position (so top slots of bracket are lined up with screws).

While holding the Adjustable VESA Bar Code holder in place, insert and tighten the bottom (2) Long Phillips Drive Flat Head Machine Screws.

Tighten top (2) Screws.

The width of the barcode holder can be adjusted by loosening two screws on the arm of the barcode holder. Once desired width is achieved, tighten (2) screws.

The adjustable barcode holder is non-handed. Simply change the orientation of the "fork" for left, right and even top mounting.

NOTE: The offset of the VESA Bar Coder Holder allows for different mounting depths of a monitor.

