

Tools Required:
5/32" Allen Wrench

Parts Included:
(1) Counterweight Plate
(4) 1/4-20 x 1" Button Head Cap Screws
(4) 1/4" Flat Washers

- 1 Align the counterweight plate to the specified holes on the underside of the base. The flat portion of the plate should be facing upward.
- 2 Using the 5/32" Allen wrench, secure the counterweight plate to the underside of the base using the 4 1/4" flat washers and the 4 1/4-20 x 1" button head cap screws

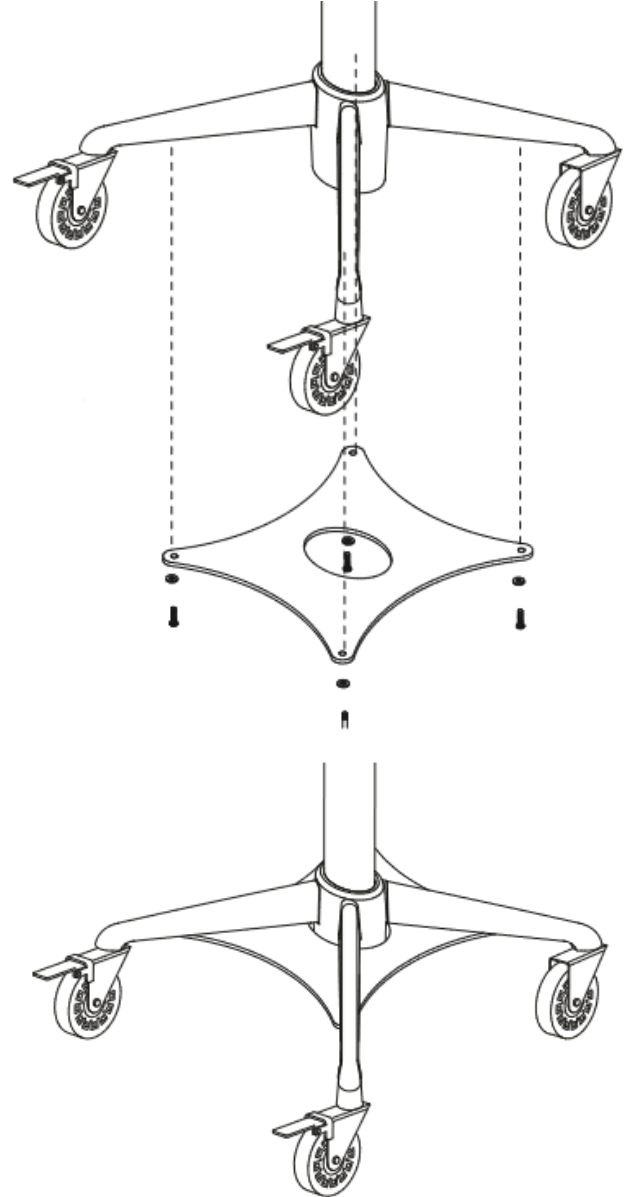


Figure 1. The counterweight being attached to the underside of the base.

If you have any questions, contact Altus at service@altus-inc.com or (616) 233-9530. If immediate assistance is needed, please call as soon as possible so we may help.